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Note: Document originally drafted in the English language.  
注释：文件最初用英语起草。

### Product Description

The G-242 is a 4.0" (102 mm) 150 kV, 445 kJ (600 kHU) maximum anode heat content, rotating anode insert. This insert is specifically designed for general radiographic, cineradiographic and fluoro/spotfilm procedures. The insert features a 14° rhenium-tungsten facing on molybdenum with a graphite backed target and is available with the following nominal focal spots:

0.4 - 0.8  
IEC 60336

Nominal Radiographic Anode Input Power  
Small - 15 kW IEC 60613  
Large - 45 kW IEC 60613

For the equivalent anode input power of 200 Watts

Continuous Anode Input Power  
490 W

**Reference Axis:**  
Perpendicular to port face.

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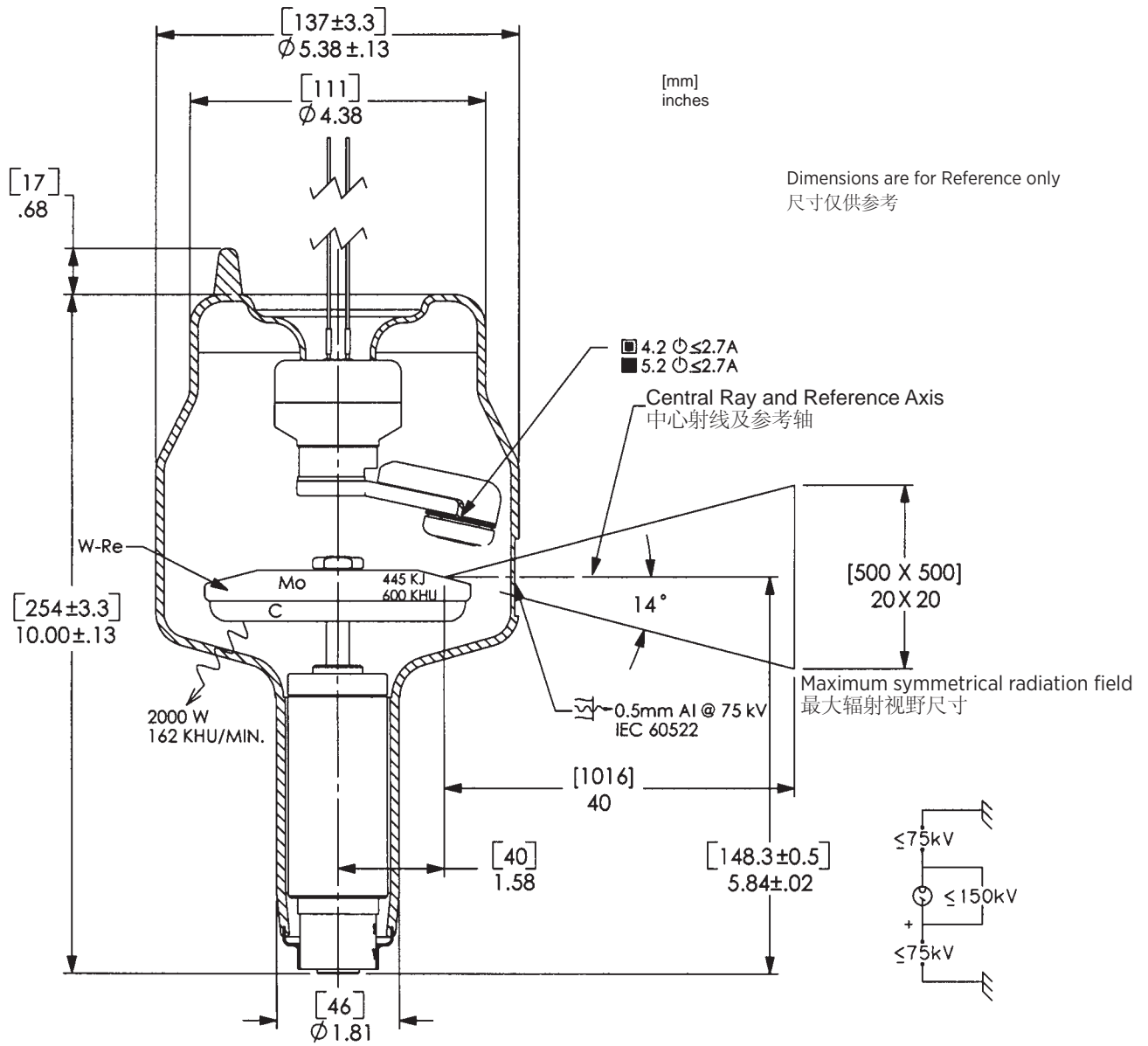
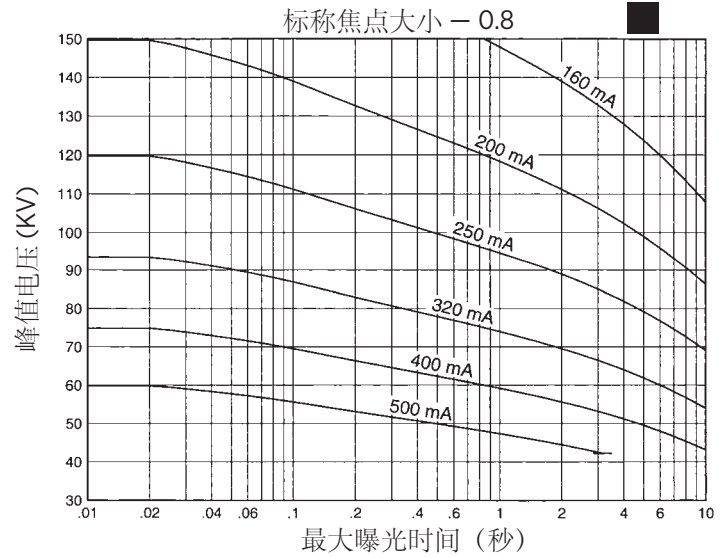
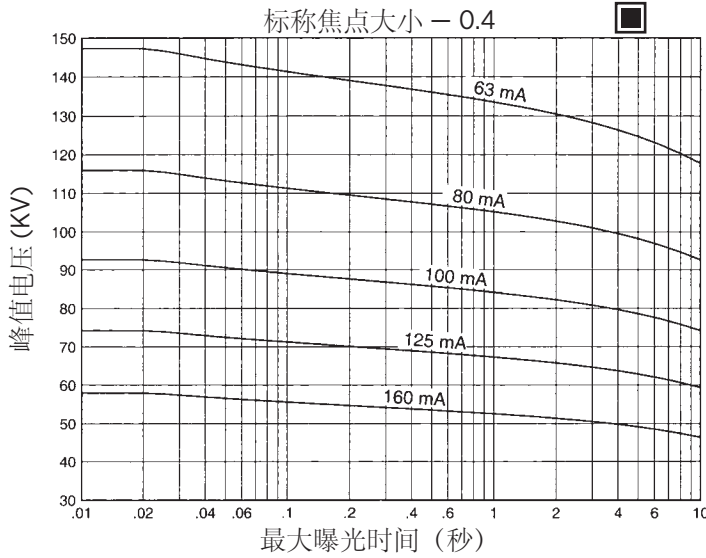
⏻ Frame or Chasis  
框架或底盘

⏻ X-Ray Tube  
X 射线管



⏻ Radiation Filter or Filtration  
辐射过滤器或过滤

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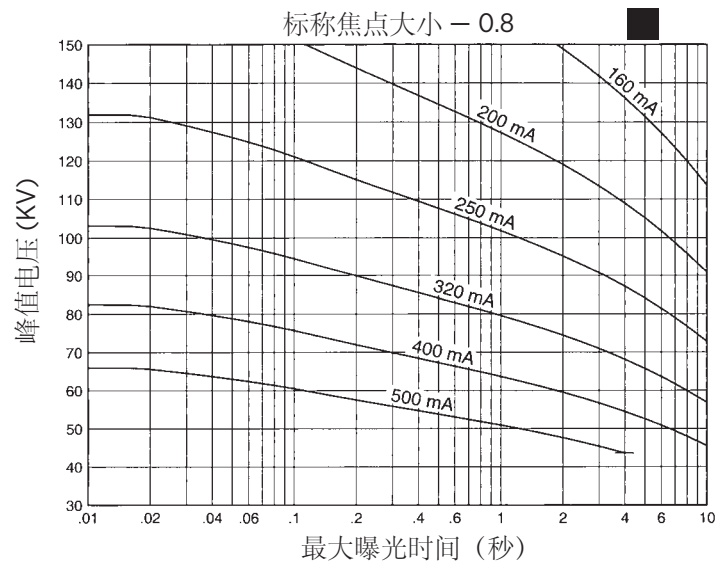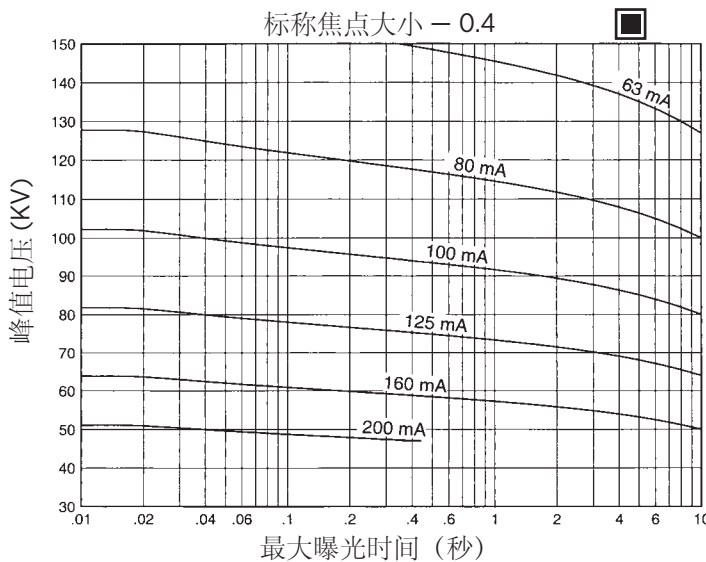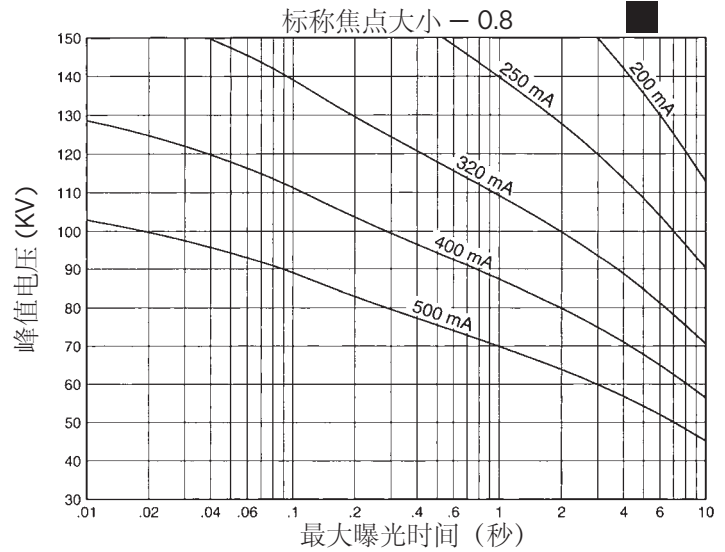
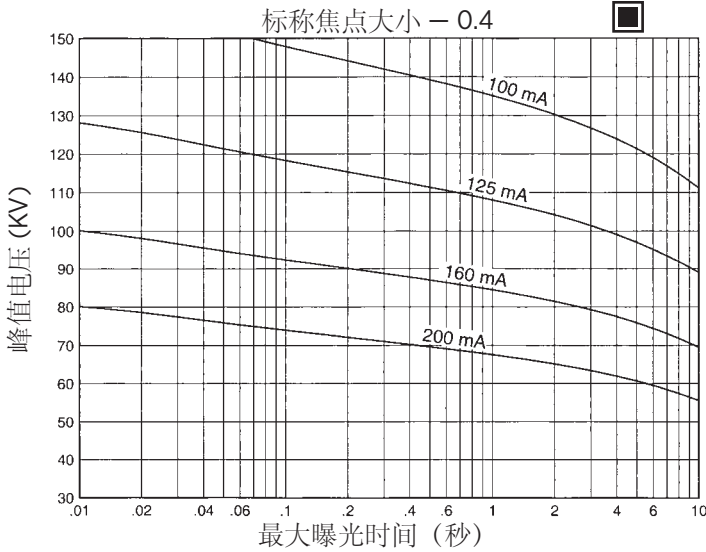
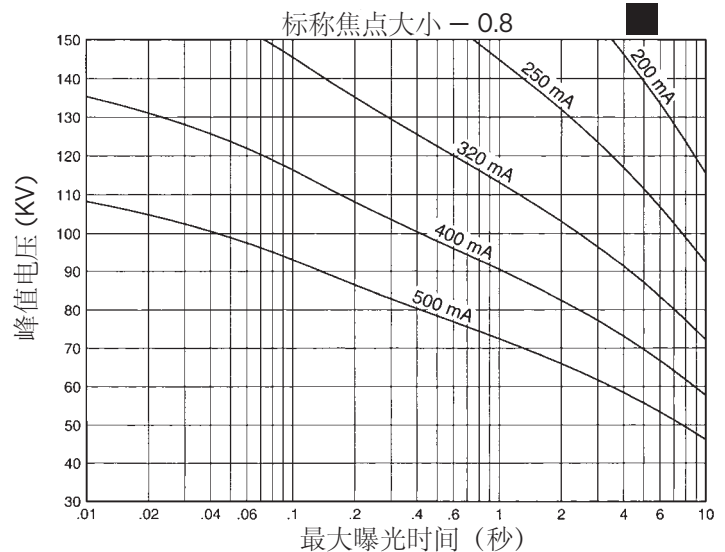
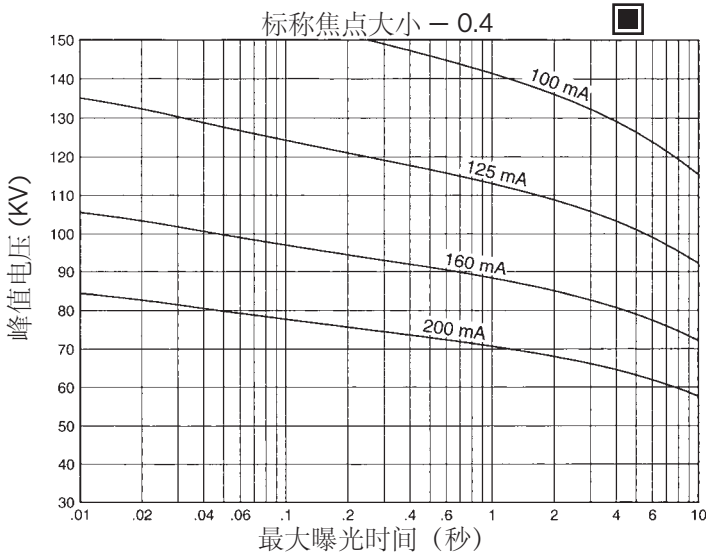
## CINERADIOGRAPHIC RATINGS HOW TO USE CINERADIOGRAPHIC CHARTS

**General:** With the Cineradiographic rating chart we can determine the maximum allowable kW of the Cine pulse, or with a given kW determine maximum time in seconds the Cine run can progress.

The Most common way of using the charts is to determine maximum time of any expected Cine run and maximum duty factor. With a known duty factor and Cine run time kW can easily be determined.

### Definition of Terms

**Time in seconds:** Total time of one Cine run, usually 5 to 12 seconds.

**Duty Factor in Percent (DF%):** Actual time during one second the x-ray tube is producing x-rays. If we select a 4 msec pulse width and 60 exposures per second the x-ray tube will be producing x-rays for a total of 240 msec each second or 24% of the time. The higher the DF number, the more load placed on the x-ray tube.

**Peak Pulse Power:** Peak energy in watts of any one Cine Pulse. Can be any combination of kV and mA allowed by Radiographic and Filament Emission curves.

Example: 80 kV at 400 mA equals

$$80,000 \text{ V} \times 0.4 \text{ A} = 32,000 \text{ W or } 32 \text{ kW}$$

### USING THE CINE RATING CHARTS:

G-242 150/180 Hz 3 Phase 0.8 Focal Spot

**Example:** Determine maximum kW allowed with the following known factors:

Maximum Pulse Width .....	4 msec
Exposures per Second .....	60
Maximum Cine Run Time .....	10 seconds

### Calculate Duty Factor: (DF%)

$$DF\% = \frac{\text{Pulse Width (mSec)} \times \text{Frames per Second}}{10}$$

$$DF\% = \frac{4 \text{ msec} \times 60 \text{ exp/sec}}{10} = \frac{240}{10} = 24\%$$

Refer to Rating Chart G-242 150/180 Hz 3 Phase 0.8 Focal Spot:

At bottom of chart find 10 second line. Move vertically to intersection with 24% DF curve. Make a horizontal reference to left side of rating chart and note kW rating of 34 kW.

We now know each pulse during the cine run can have a maximum rating of 34 kW under conditions given in example.

kW = kV x mA. The kW of the exposure can be any combination of mA and kV allowed by the Radiographic and Filament Emission Charts.

The Cine rating charts are usable to 100% anode heat storage. The start of Cine run should be below 70% anode heat storage. Exceeding 100% anode heat storage will cause anode track erosion with high risk of tube destruction.

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## SERIAL LOAD RATINGS HOW TO USE SERIAL LOAD RATING CHARTS

**General:** Serial Radiography puts a severe demand on the x-ray tube due to the large number of exposures made in rapid succession. Intervals between exposures are fixed and so short that it is not possible for the anode track to cool to any extent during the exposure series. Therefore, the temperature of the anode track increases from exposure to exposure. The kW values used in the angiographic charts have been determined to prevent damage to the anode. The angiographic rating charts are usable to 100% anode heat storage. Exceeding 100% anode heat storage will cause anode track erosion with high risk of tube destruction.

### Definition of Terms

**Number of Exposures in Series:** The number of exposures made in succession or the number of exposures made during one contrast injection.

**Exposure Rate:** The number of exposures made per second. For a series of exposures where the exposure rate changes, it must be assumed that all exposures will be made at the maximum rate. For example, if during a series 10 exposures will occur at one per second and 30 exposures at 4 per second use the kW ratings in the 40 exposure column at 4 per second rate.

**Exposure Time:** Time in seconds of each exposure.

### USING THE CHARTS:

#### Select Correct Chart:

50/60 or 150/180 Hz  
0.4 or 0.8 Focal Spot

**Note:** 150/180 Hz rotor speed recommended for all angiography.

**Determine the number of exposures in Series:** With cut film angiography the number of exposures are known, however in Digital Angiography the number of exposures commonly are not known. When determining the number of exposures, assume worst case or past history.

**Note:** Most angiographic x-ray tubes fail from under-estimating the number of exposures made in a series.

**Determine kW of each exposure in Series:** Referring to chart —find block under “Number of Exposures in Series” that is greater than or equal to expected number of exposures in Series. On left side directly opposite this block under “Exposure Rate per Second” column, select maximum rate per second that will be used for the exposure series. At the intersection of exposure rate and exposure time in seconds, find maximum kW allowed for each exposure.

**kW = pkV x mA:** The kW of the exposure can be any combination of mA and pkV allowed by the Radiographic and Filament Emission charts.

For Example: 80 pkV and 500 mA = 40 kW

**Example:** From chart G-242 150/180 Hz 3 Phase  
0.8 Focal Spot, determine kW allowed with  
following known factors.  
Maximum number of exposures ..... 40  
Exposure time .050 second (50 milliseconds)  
Maximum Exposures per second ..... 4

From chart find 40 exposure block. On left side directly opposite this block under “Exposure Rate per Second” column, select 4 exposures per second. Find .050 seconds at top of chart. At intersection of exposure rate line and exposure time, find 33.8 kW.

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
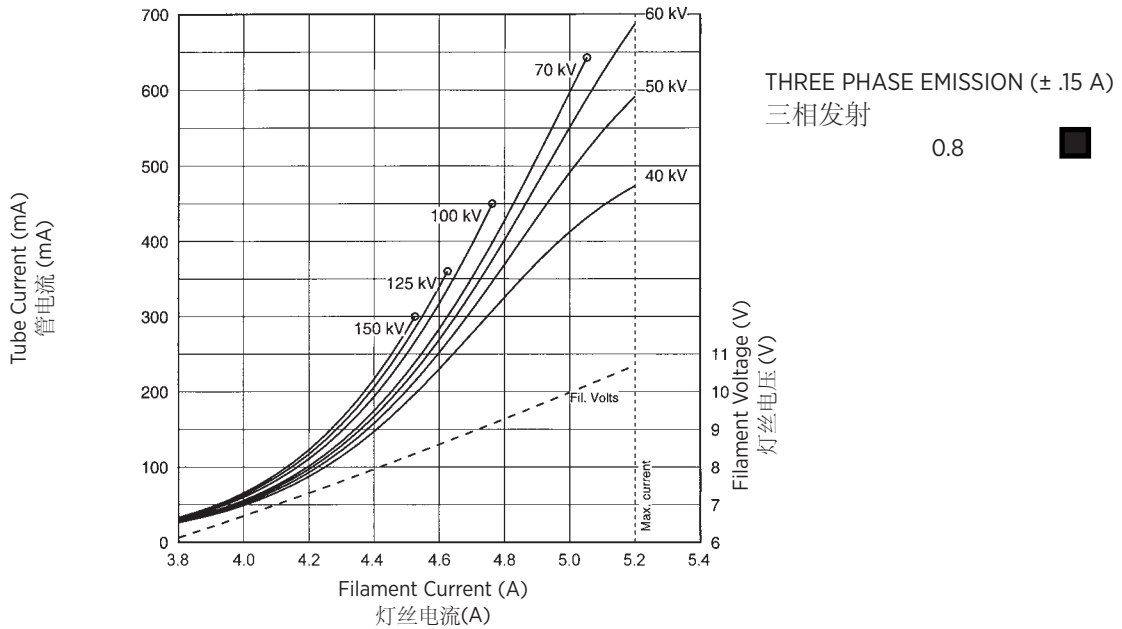
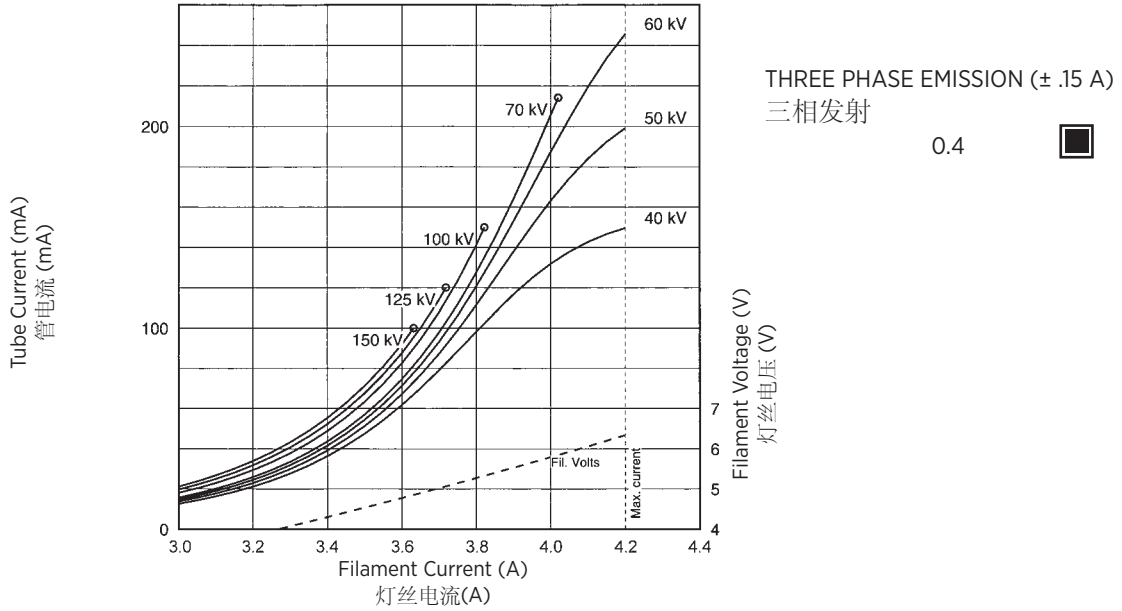
Note: When using these emission curves for trial exposures, refer to the power rating curves shown for maximum kV, tube emission, filament current, exposure time, and target speed.

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**ANODE HEATING AND COOLING CURVES**

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





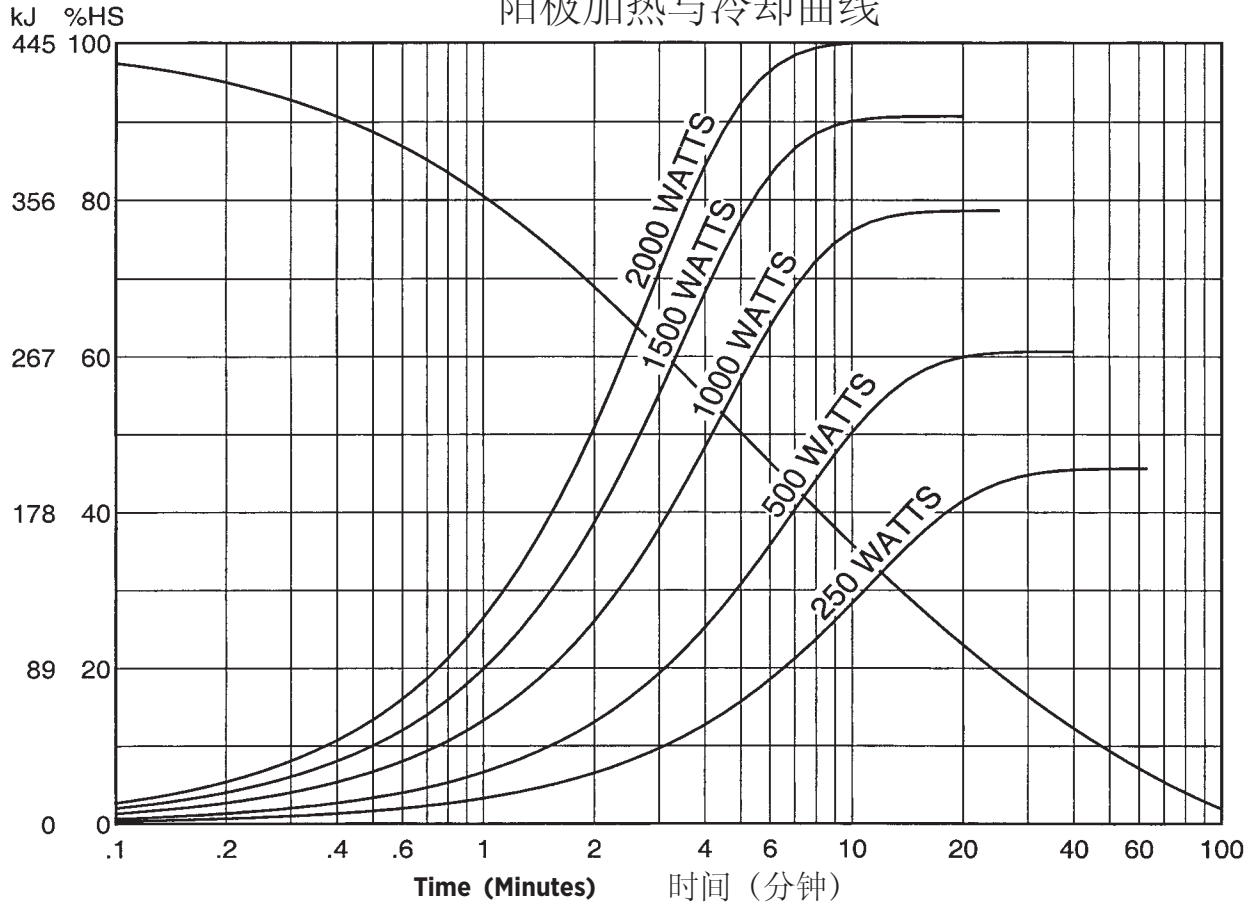
Product Description	
Nominal X-ray Tube Voltage	150 kV
Maximum Cathode to Ground	75 kV
Maximum Anode to Ground	75 kV
Maximum X-ray Tube Assembly Heat Content	1480 kJ (2,000 kHU)
Nominal Continuous Input Power (max. housing temperature 78°C) (Includes all sources)	2.0 kW (2.7 kHU/sec) IEC 60613:2010
X-Ray Tube Assembly Permanent Filtration	0.7 mm Al/75 kV IEC 60522/1999
High Voltage Cable Receptacles	Per IEC 60526
Loading Factors for Leakage Radiation	150 kV, 13.3 mA
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to +75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Weight, Approximate:	
Housing	21.4 kg (47 lbs)
HE	6.01 kg (13.0 lbs. 4 oz.) dry
Stator Cord	6.1 M (20 Feet)
Thermal Switch	Normally Closed 7A @ 120Vac or 30 Vdc max.
Open	85°C ±3.9°C (185°F ±7°F)
IEC Classification	Class 1
X-Ray Tube Assembly (Complies to)	IEC 60601-2-28

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
### X-ray Tube Assembly Heating and Cooling Curve

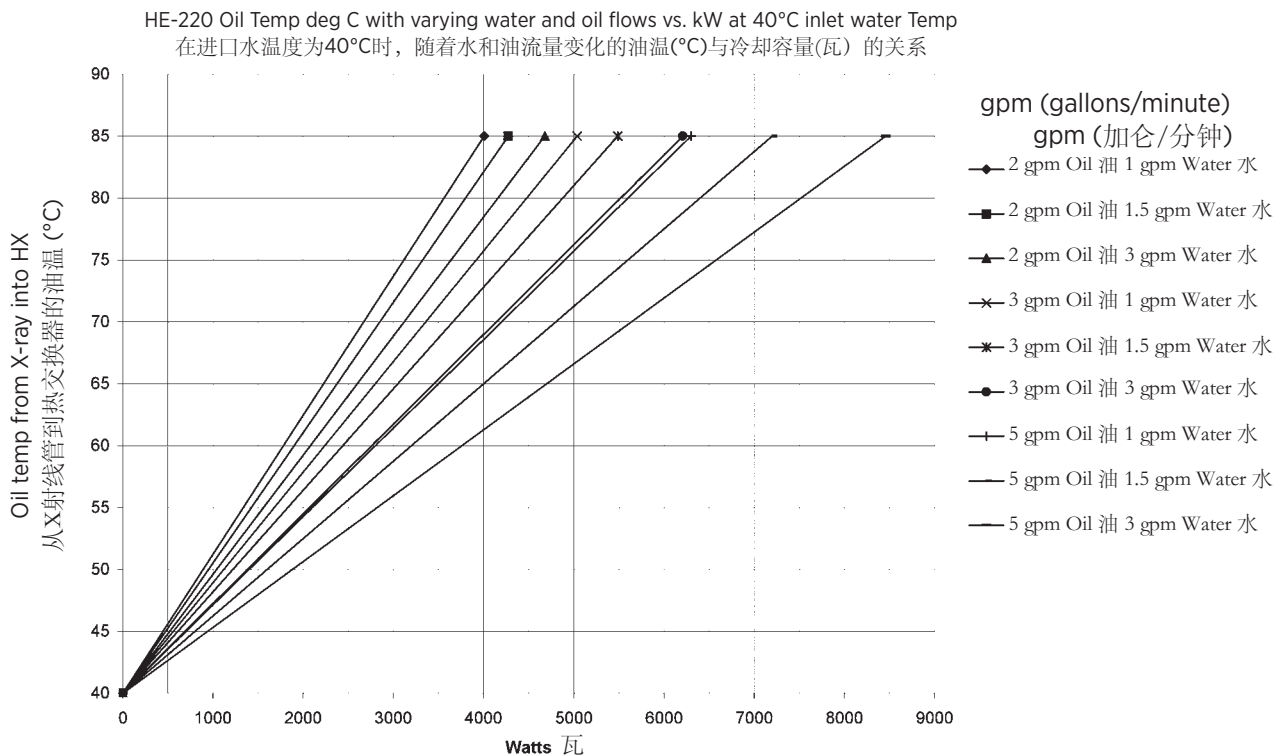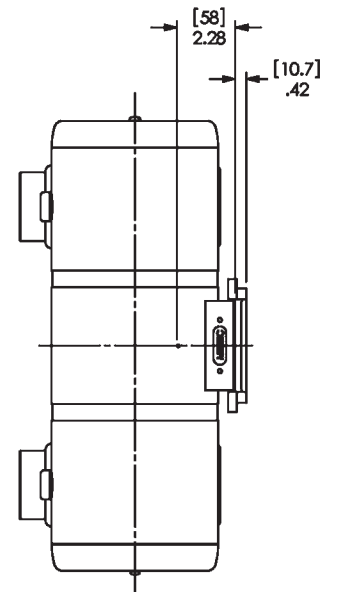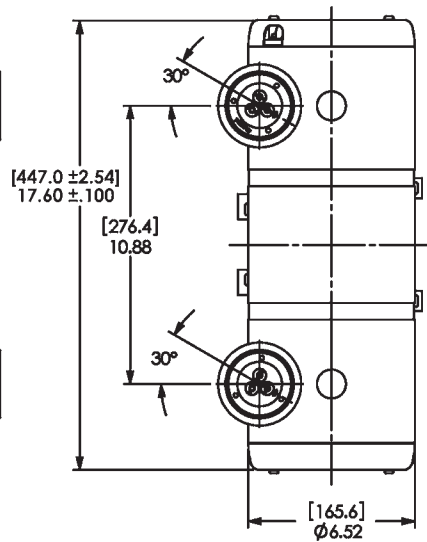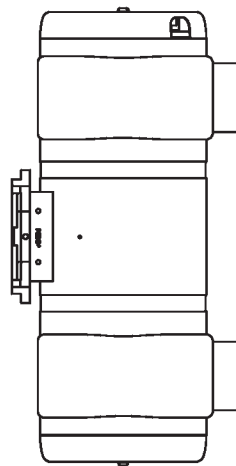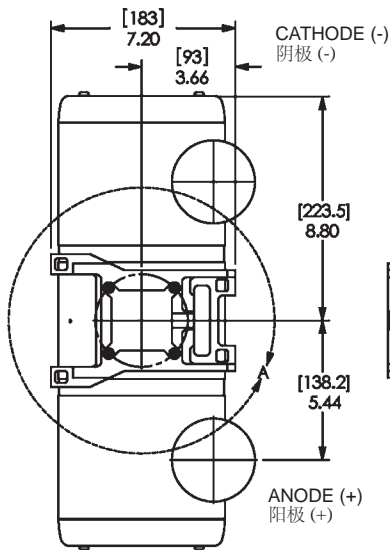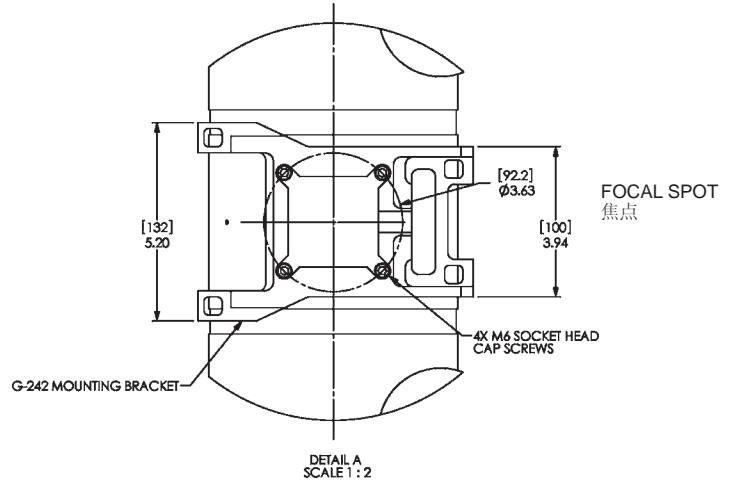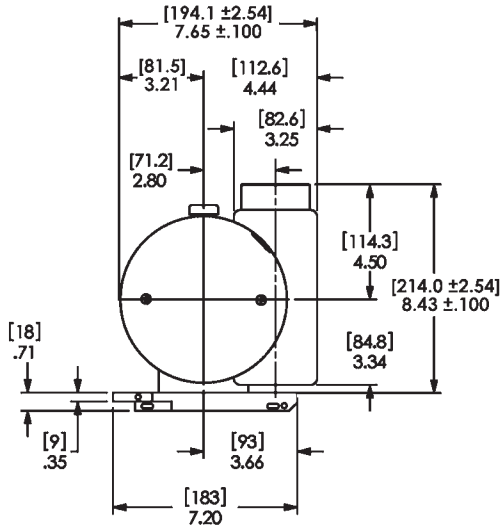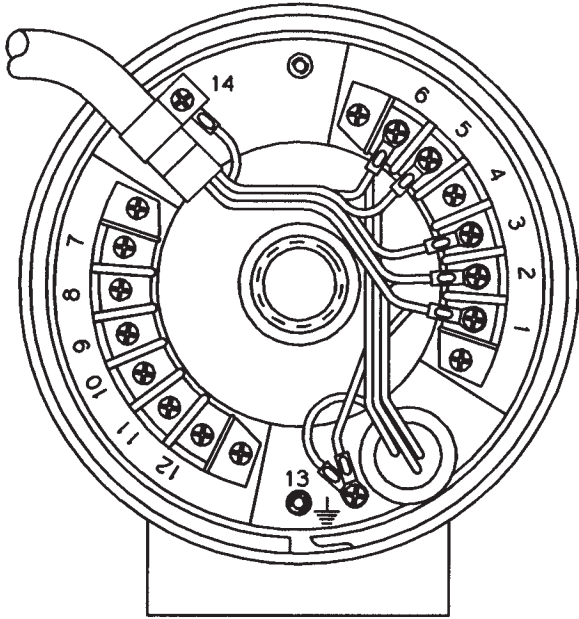
Dimensions are for reference only  
尺寸仅供参考

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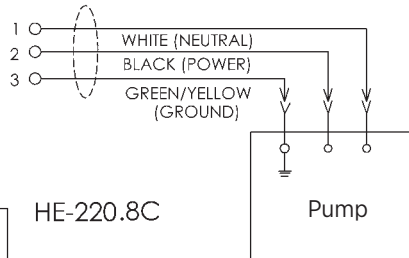**Stator Power:**  
Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 102 mm (4 inch) anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible. The starting voltage must never exceed 600 volts rms.

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Heat Exchanger (Oil to Water)  
热交换器(油到水)

**DESCRIPTION:**

Heat Exchanger (oil to water).  
For use with B-130H housings.

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**PERFORMANCE:**

2.0 kW with the following conditions:  
1) 2 GPM Water Flow (36°C)

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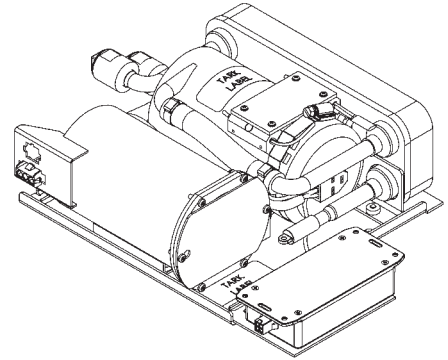
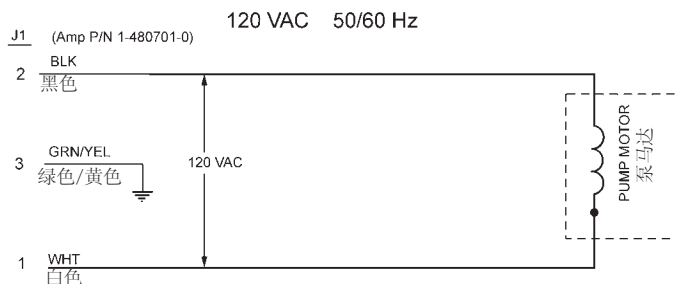
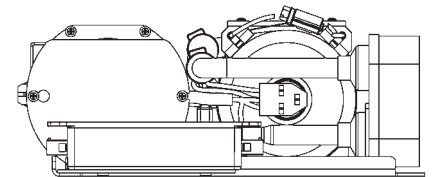
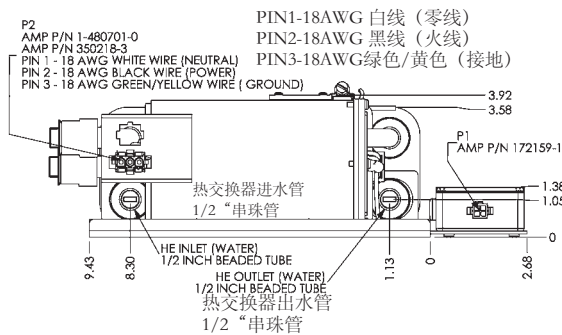
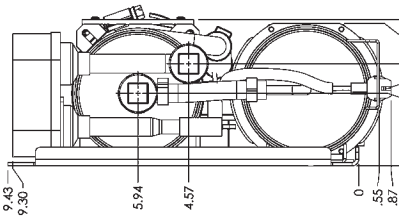
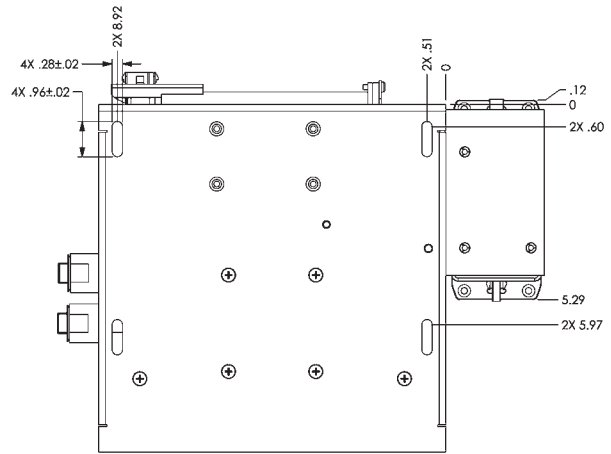
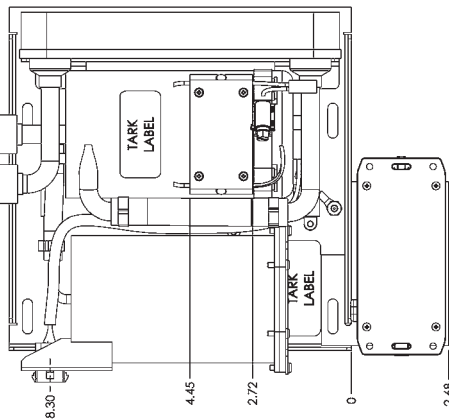
Manufactured by Varex Imaging Corporation  
由万睿视影像有限公司生产

Specifications subject to change without notice.  
规格如有更改，恕不另行通知。