Rotating Anode X-Ray Tube
 Tubes Radiogènes à Anode Tournante
 Röntgenröhre mit rotierender Anode
 Tubos de Rayos-X con Ánodo Giratorio

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

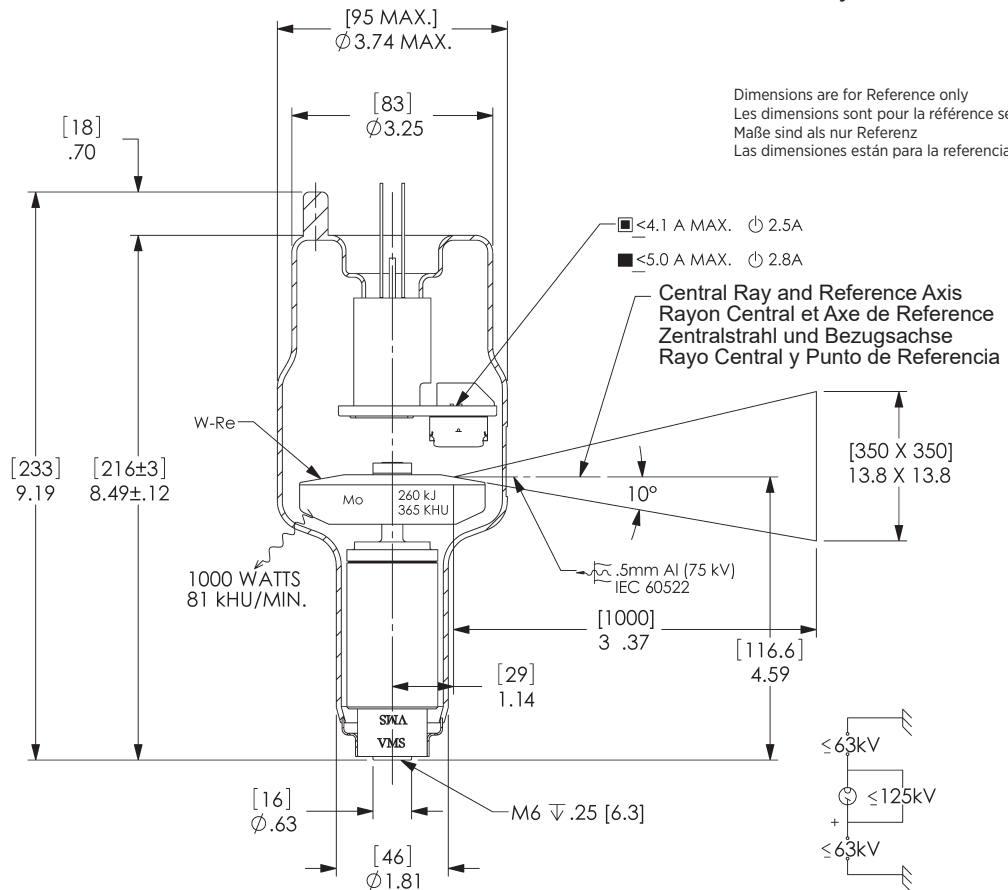
Note: Document originally drafted in the English language.

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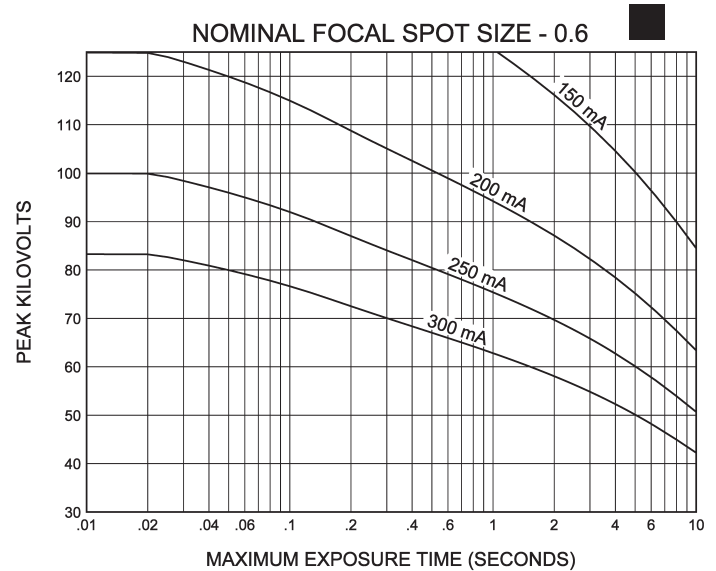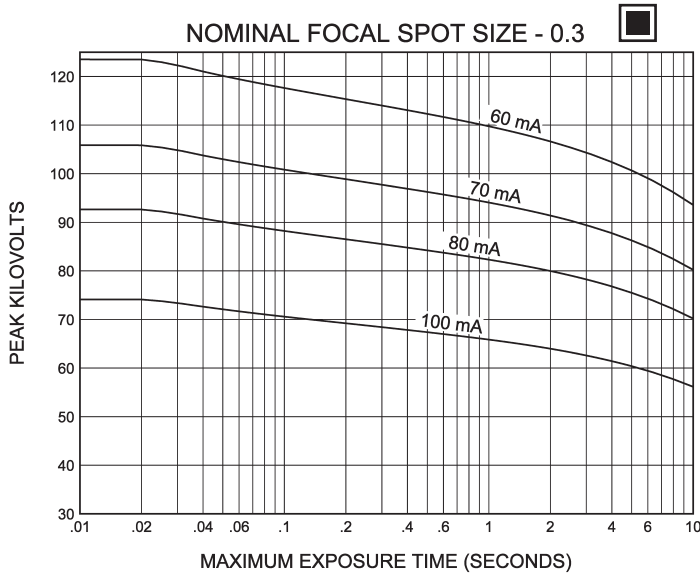
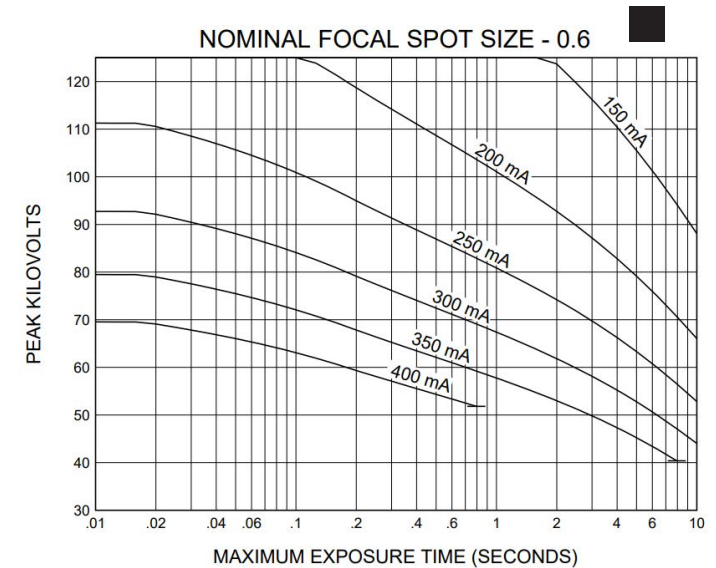
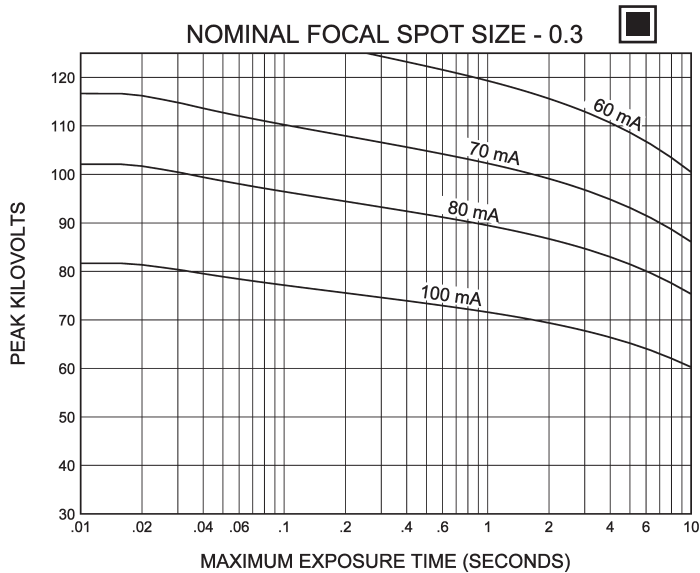
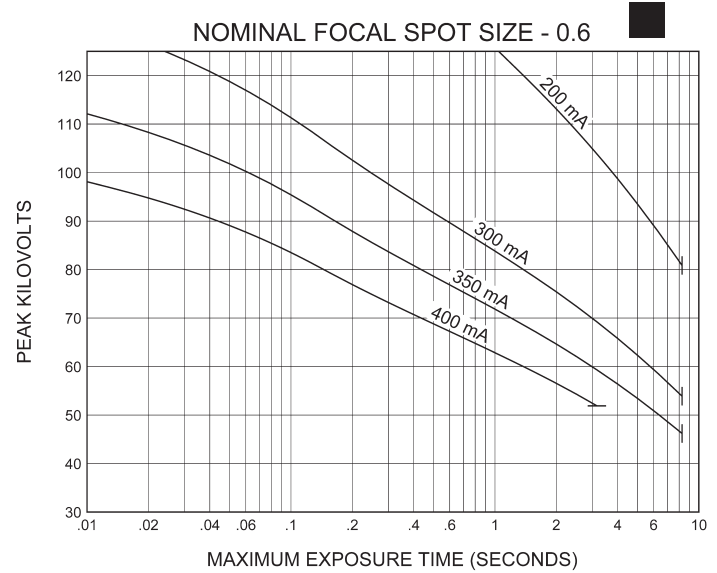
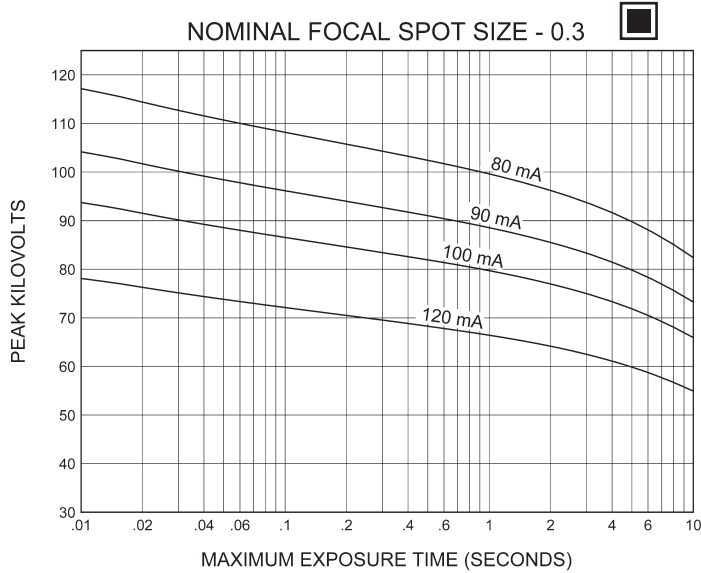
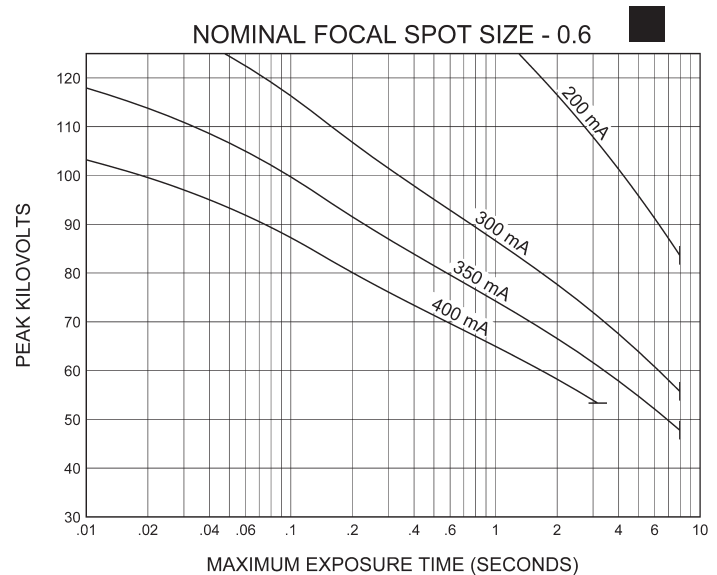
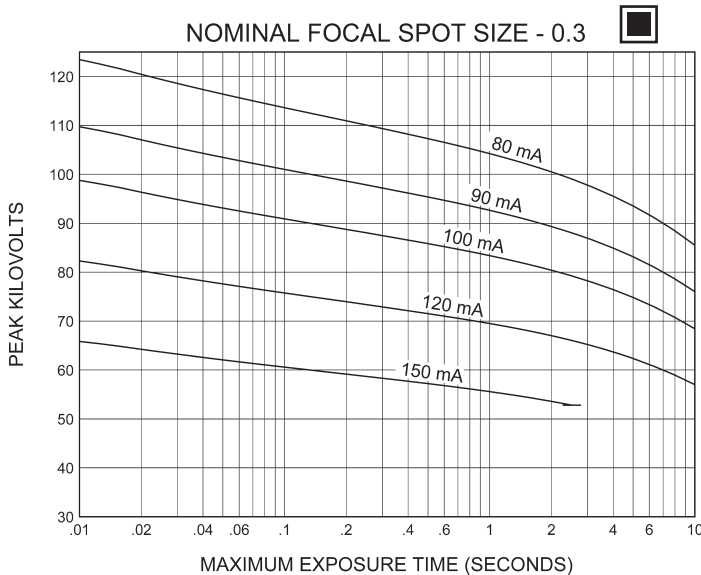
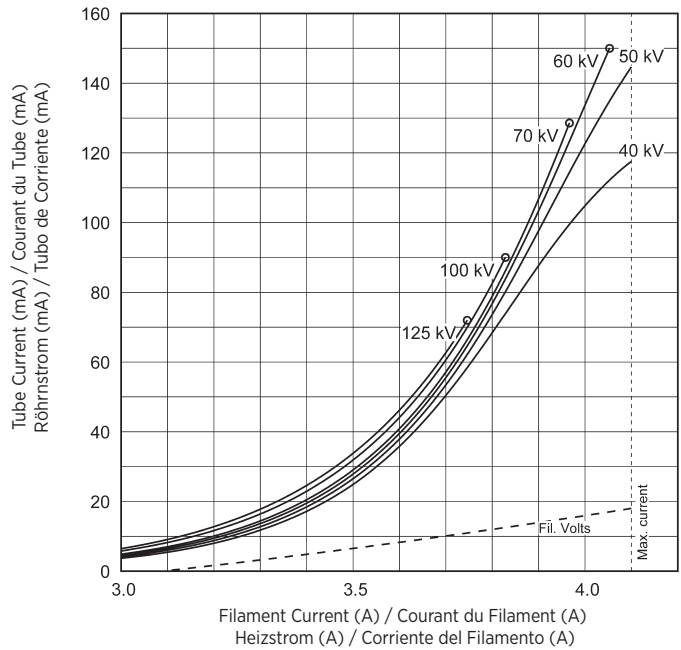
THREE PHASE EMISSION ($\pm .15$ A)
 0.3

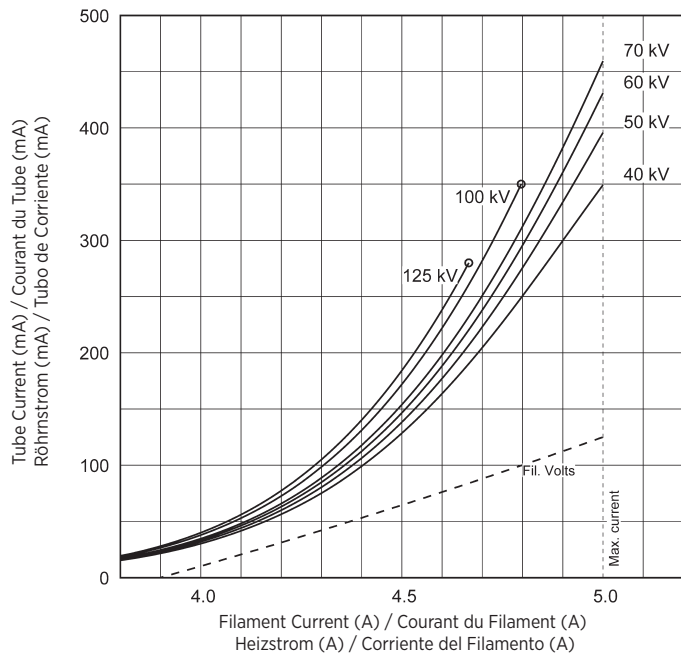THREE PHASE EMISSION ($\pm .15$ A)
 0.6

Note: When using these emission curves for trial exposures, refer to the power rating curves shown for maximum kV, tube emission, filament current, exposure time, and target speed.

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CINERADIOGRAPHIC RATINGS

HOW TO USE CINERADIOGRAPHIC CHARTS

General: With the Cineradiographic rating chart we can determine the maximum allowable kW of the Cine pulse, or with a given kW determine maximum time in seconds the Cine run can progress.

The Most common way of using the charts is to determine maximum time of any expected Cine run and maximum duty factor. With a known duty factor and Cine run time kW can easily be determined.

Definition of Terms

Time in seconds: Total time of one Cine run, usually 5 to 12 seconds.

Duty Factor in Percent (DF%): Actual time during one second the x-ray tube is producing x-rays. If we select a 4 msec pulse width and 60 exposures per second the x-ray tube will be producing x-rays for a total of 240 msec each second or 24% of the time. The higher the DF number, the more load placed on the x-ray tube.

Peak Pulse Power: Peak energy in watts of any one Cine Pulse. Can be any combination of kV and mA allowed by Radiographic and Filament Emission curves.

Example: 80 kV at 400 mA equals
 $80,000 \text{ V} \times 0.4 \text{ A} = 32,000 \text{ W}$ or 32 kW

USING THE CINE RATING CHARTS:

RAD-15 60 Hz 3 Phase 0.6 Focal Spot

Example: Determine maximum kW allowed with the following known factors:

- Maximum Pulse Width 4 msec
- Exposures per Second 60
- Maximum Cine Run Time ... 10 seconds

Calculate Duty Factor: (DF%)

$$DF\% = \frac{\text{Pulse Width (mSec)} \times \text{Frames per Second}}{10}$$

$$DF\% = \frac{4 \text{ msec} \times 60 \text{ exp/sec}}{10} = \frac{240}{10} = 24\%$$

Refer to Rating Chart

RAD-15 60 Hz 3 Phase 0.6 Focal Spot:

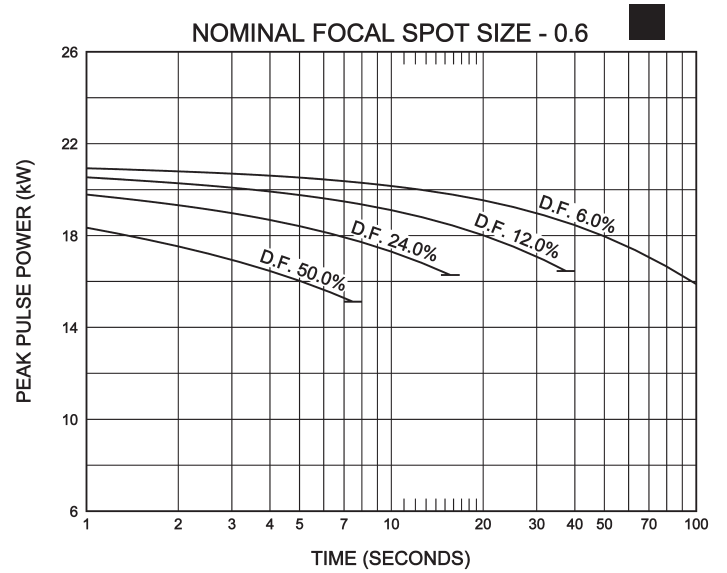
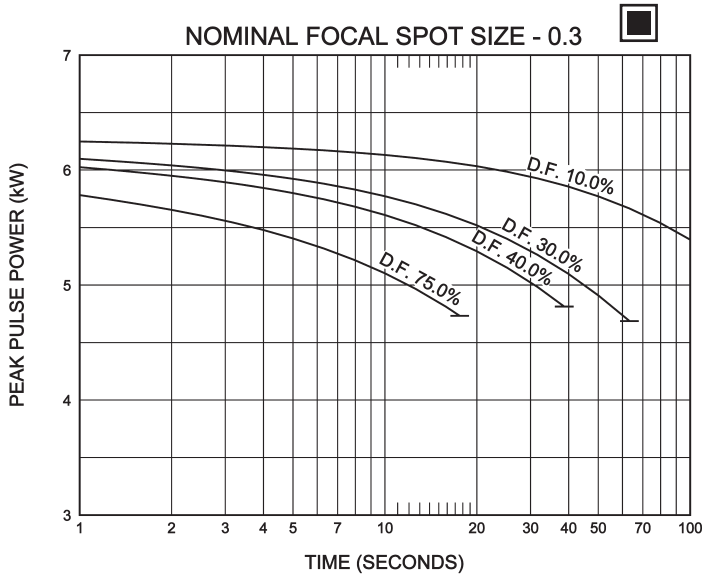
At bottom of chart find 10 second line. Move vertically to intersection with 24% DF curve. Make a horizontal reference to left side of rating chart and note kW rating of 19 kW.

$kW = kV \times mA$. The kW of the exposure can be any combination of mA and kV allowed by the Radiographic and Filament Emission Charts.

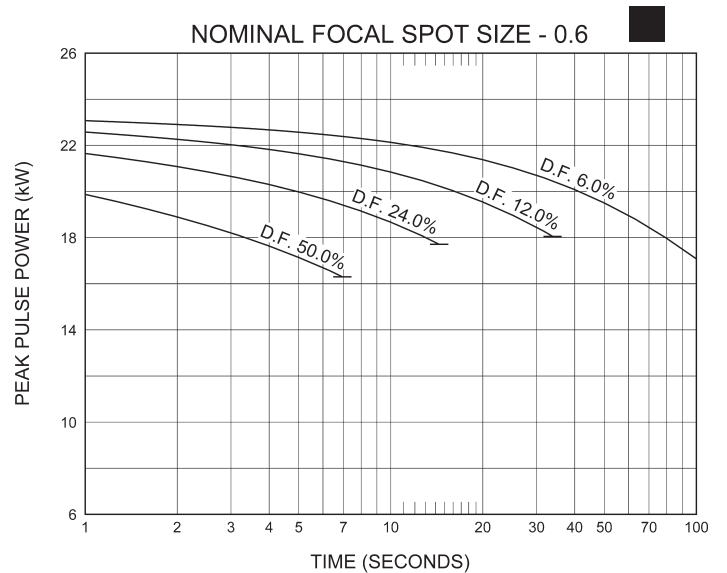
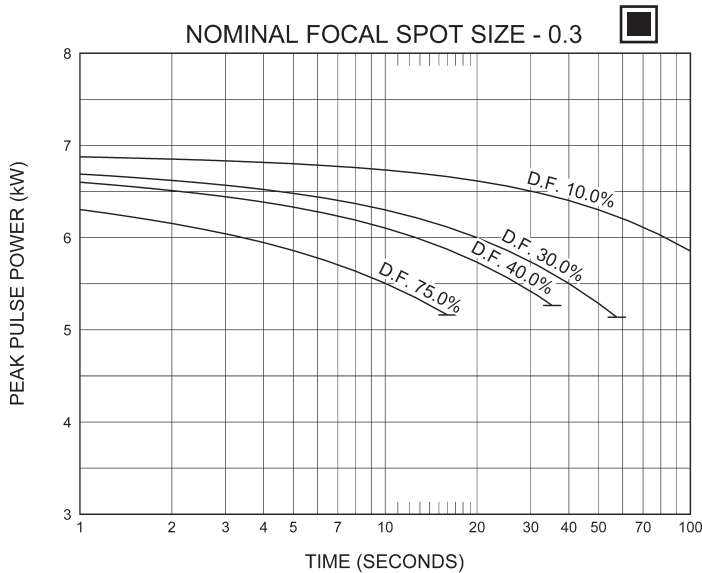
The Cine rating charts are usable to 100% anode heat storage. Exceeding 100% anode heat storage will cause anode track erosion with high risk of tube destruction.

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