Note: Document originally drafted in the English language.

Product Description

The B-115 housing is designed for Varex Imaging mammography inserts.

IEC Classification Class 1
 Weight, Approximate:
 Housing 10.2 kg (22.5 lbs)
 Stator Cord 4.5 m (14 Ft)
 Mounting Port
 X-Ray Tube Assembly (Complies to) IEC 60601-2-28

Caution: Tubes ran in the vertical position must have the two thermal switch configuration.

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Product Description	
Nominal X-ray Tube Voltage	
Maximum Cathode to Ground	0 kV
Maximum Anode to Ground	39 - 49 kV
Grid Control Voltages: (If Applicable)	
Typical Bias Voltage for Focals	-5 to -100 Vdc
Maximum X-Ray Tube Heat Content	370 kJ (500 kHU)
Nominal Continuous Input Power	
Horizontal	100 Watts (135 HU/sec) IEC 60613:2010
Vertical	90 Watts (121.5 HU/sec) IEC 60613:2010
Maximum Housing Temperature	78°C
X-Ray Tube Assembly	
Permanent Filtration	0.63 mm Be IEC 60522/1999 (0.0mm Al/49 kV)
Leakage Technique Factors	49 kV, 2.0 mA
Cable Receptacles	Non Federal Standard 55 kV - Rating
Thermal Switch	normally closed
Open	79.4°C ±3.9°C (175°F ±7°F) Rating - 10A @ 240VAC
Second Thermal Switch for Vertical Applications	85°C ±3°C (185°F ±5.4°F) Rating - 10A @ 240VAC
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transportation	-10°C to 75°C
Humidity	10% to 90%
Atmospheric Pressure Range	70 kPa to 106 kPa

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Note 1 Dimensions listed are for the standard port boss
 Metric M6 X 1.0 available upon request

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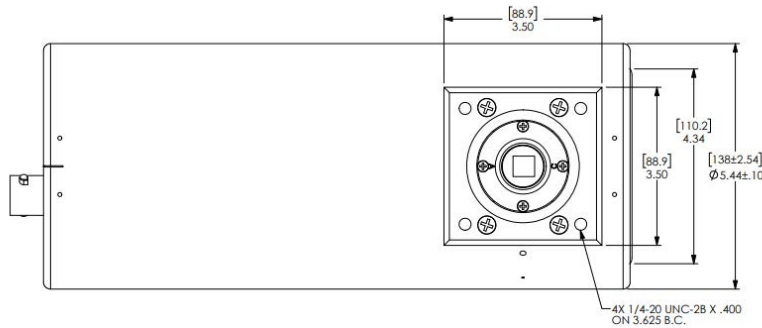
EXTENDED PORT BOSS
 DIM. F = 3.20 INCHES [81.3mm]
 DIM. I = 1.89 INCHES [48mm]

BIANGULAR INSERTS
M-103T, M-113 Series, M-171

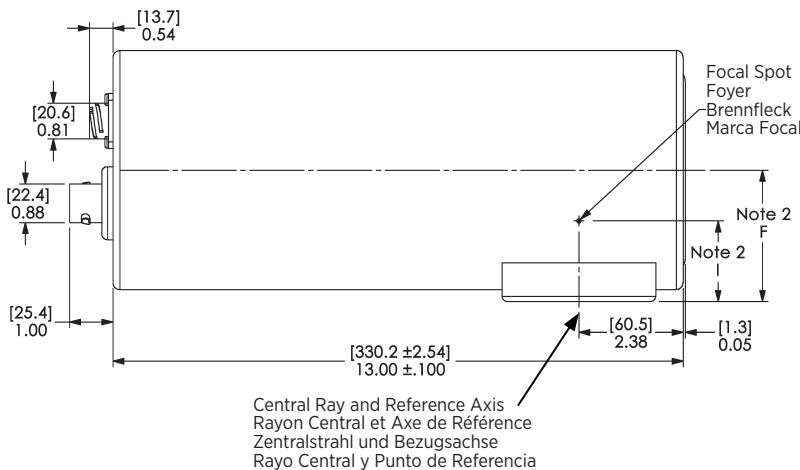
STANDARD/METRIC PORT BOSS
 DIM. F = 2.97 INCHES [75.4mm]
 DIM. I = Refer to Drawing

EXTENDED PORT BOSS
 DIM. F = 3.20 INCHES [81.3mm]
 DIM. I = Refer to Drawing

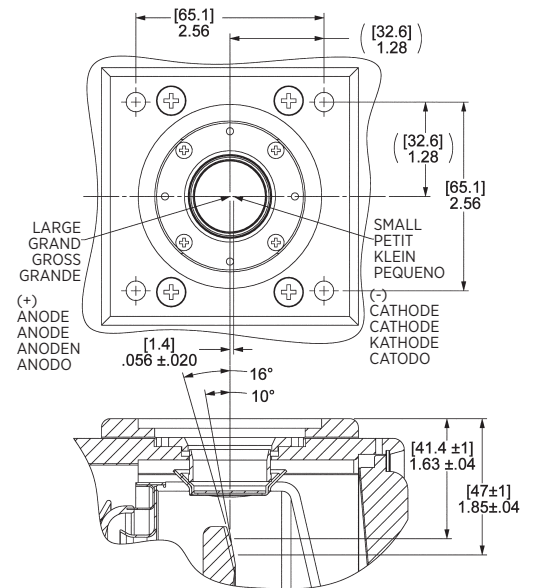
Note: Dimensions in [mm] Inches
 Remarque: Dimensions en [mm] Pouces
 Hinweis: Abmessungen in [mm] Zoll
 Nota: Dimensiones en [mm] Pulgadas

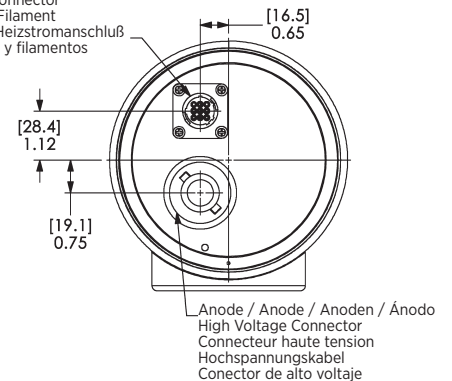
FRONT VIEW
 VUE DE FACE
 VONDERANSICHT
 DELANTER EXPUESTA

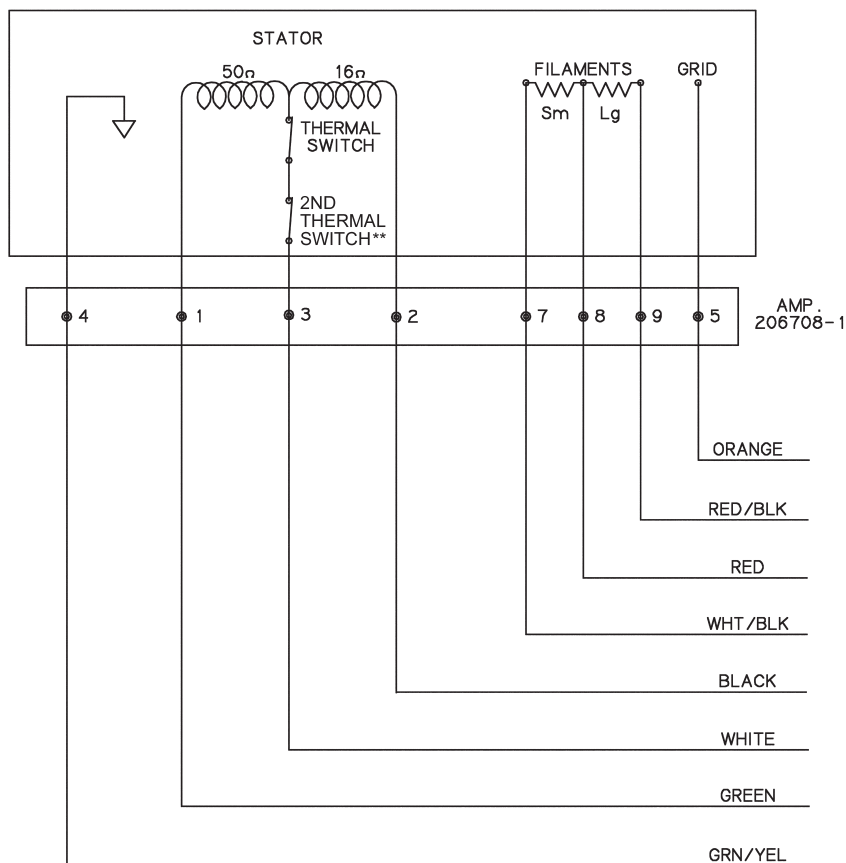
BIANGULAR INSERTS

Cathode / Cathode / Kathode / Cátodo
 Stator and Filament Connector
 Connecteur Stator et Filament
 Statoranschluß und Heizstromanschluß
 Conector de la Bovina y filamentos

Stator Ratings and Characteristics
 Spécificités et Caractéristiques du Stator
 Statornennleistungen und Merkmale
 Características y Clarificación de la Bovina

1 Green	Phase Shift
Vert	Changement de Phase
Grün	Hilfsphase
Verde	Cambio de Fase del Estator
2 Black	Phase
Noir	Phase
Schwarz	Phase
Negro	Fase
3 White	Common
Blanc	Neutre
Weiss	Neutral
Blanco	Común
4 Green/Yellow	Housing Ground
Vert/Jaune	Masse de la Gaine
Grün/Gelb	Masse des Gehäuses
Verde/Amerillo	Encaje a Tierra
5 Orange	Grid
Orange	Grille
Orange	Gitter
Anaranjado	Rejillas
7 White/Black	Small Filament
Blanc/Noir	Filament Petit
Weiss/Schwarz	Glüfaden Klein
Blanco/Negro	Filamento Pequeño
8 Red	Common Filament
Rouge	Filament Neutre
Rot	Glüfaden Neutral
Rojo	Filamento Común
9 Red/Black	Large Filament
Rouge/Noir	Filament Grand
Rot/Schwarz	Glüfaden Gross
Rojo/Negro	Filamento Largo

** Second thermal switch for vertical applications only, installed at top of housing (cathode side)
 ** Second interrupteur thermique pour les applications verticales uniquement, installé sur le dessus du boîtier (côté cathode)
 ** Zweiter Thermoschalter nur für vertikale Anwendungen, installiert an der Oberseite des Gehäuses (Kathodenseite)
 ** Segundo interruptor térmico sólo para aplicaciones verticales, instalado en la parte superior de la carcasa (lado del cátodo)

“STD” Stators	Stator “STD”	“STD” Stator	“STD” Bovina	
Black/White	Noir/Blanc	Schwarz/Weiss	Negro/Blanco	16 Ω
Black/Green	Noir/Vert	Schwarz/Grün	Negro/Verde	66 Ω
Green/White	Vert/Blanc	Grün/Weiss	Verde/Blanco	50 Ω
180 Hz Cap	180 Hz Cap	180 Hz Cap	180 Hz Cap	6 μF
60 Hz Cap	60 Hz Cap	60 Hz Cap	60 Hz Cap	30 μF

Stator Drive Frequency Fréquence d'entraînement du stator Statorantrieb Frequenz Frecuencia de la impulsión del estator	RPM
50 Hz	2800 - 3000
60 Hz	3400 - 3600
150 Hz	8500 - 9000
180 Hz	9500 - 10,800

Stator Power:
 Time to full speed of the anode is a function of the power rating of the “starter” and the weight/diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 71mm and 77mm anode series tubes is between 1.3 and 2.0 seconds.
 Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.
 No more than two high speed starts per minute are permissible.
Starter:

	Start	Run
50/60 Hz	115 VAC	45 VAC
150/180 Hz	230 VAC	90 VAC

NOTE:
 For non gridded operation connect Grid Pin (Pin 5) to Filament Common (Pin 8).

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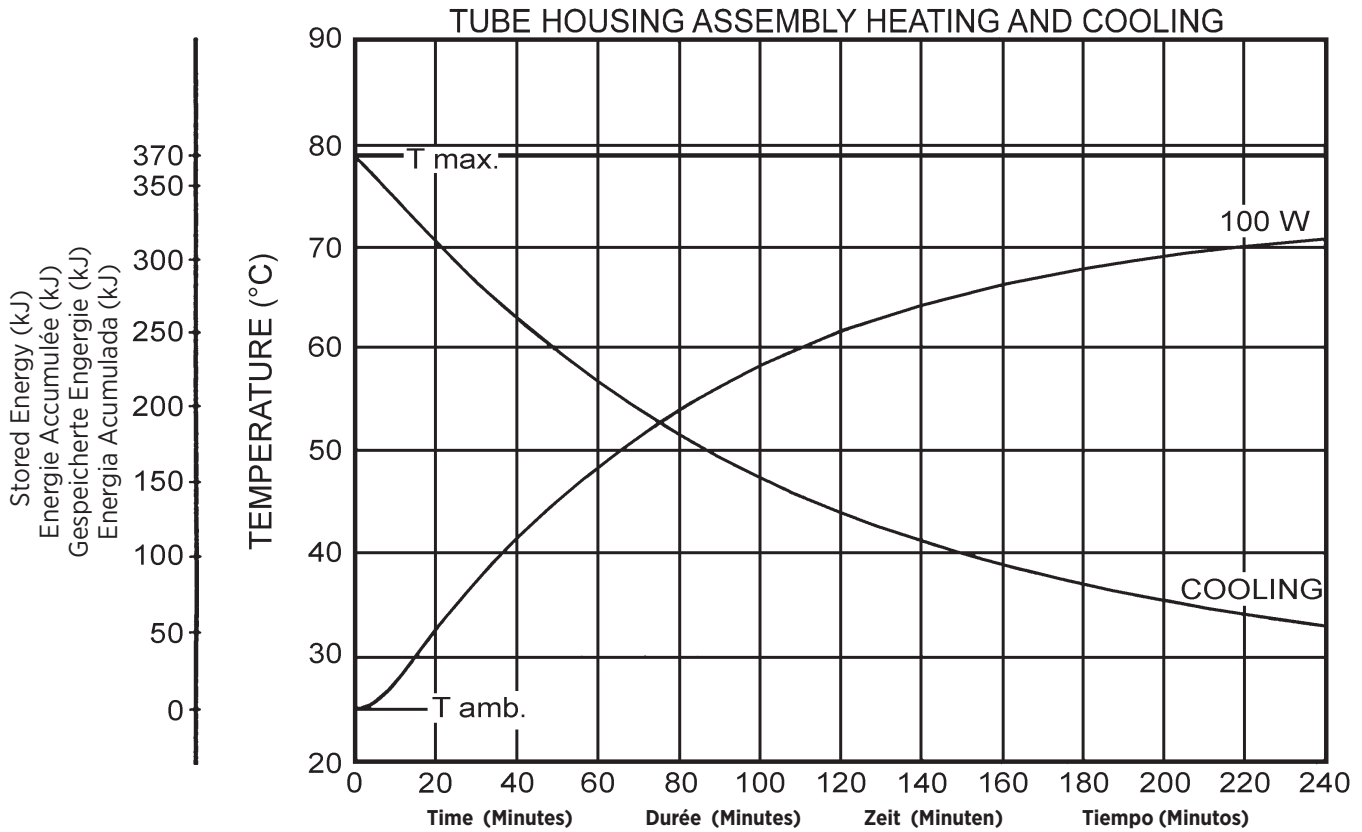
Note:
 Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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