Note: Document originally drafted in the English language.
注释：文件最初用英语起草。

Product Description

The B-115 housing is designed for Varex Imaging mammography inserts.

IEC Classification Class 1

Weight, Approximate:

Housing 10.2 kg (22.5 lbs)

Stator Cord 4.5 m (14 Ft)

Mounting Port

X-Ray Tube Assembly (Complies to).. IEC 60601-2-28

Caution: Tubes ran in the vertical position must have the two thermal switch configuration.

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<u>Product Description</u>	
Nominal X-ray Tube Voltage	
Maximum Cathode to Ground	0 kV
Maximum Anode to Ground	39 - 49 kV
Grid Control Voltages: (If Applicable)	
Typical Bias Voltage for Focals	-5 to -100 Vdc
Maximum X-Ray Tube Heat Content	370 kJ (500 kHU)
Nominal Continuous Input Power	
Horizontal	100 Watts (135 HU/sec) IEC 60613:2010
Vertical	90 Watts (121.5 HU/sec) IEC 60613:2010
Maximum Housing Temperature	78°C
X-Ray Tube Assembly	
Permanent Filtration	0.63 mm Be IEC 60522/1999 (0.0mm Al/49 kV)
Leakage Technique Factors	49 kV, 2.0 mA
Cable Receptacles	Non Federal Standard 55 kV - Rating
Thermal Switch	
Open	normally closed 79.4°C ±3.9°C (175°F ±7°F) Rating - 10 A @ 240VAC
Second Thermal Switch for Vertical Applications ..	85°C ±3°C (185°F ±5.4°F) Rating - 10A @ 240VAC
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transportation	-10°C to 75°C
Humidity	10% to 90%
Atmospheric Pressure Range	70 kPa to 106 kPa

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



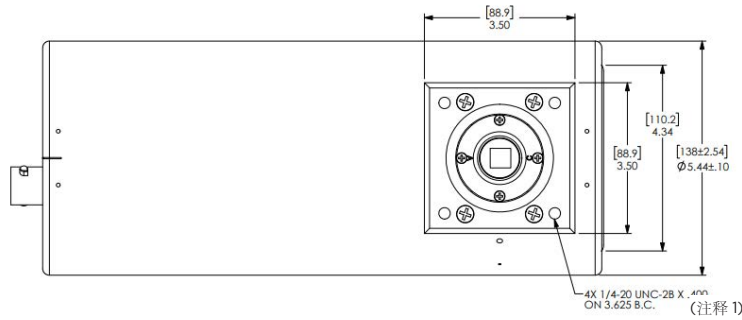
FRONT VIEW
看前部

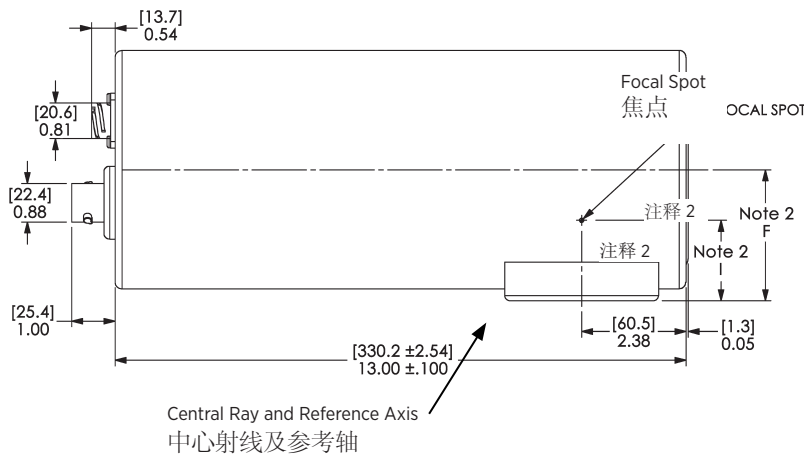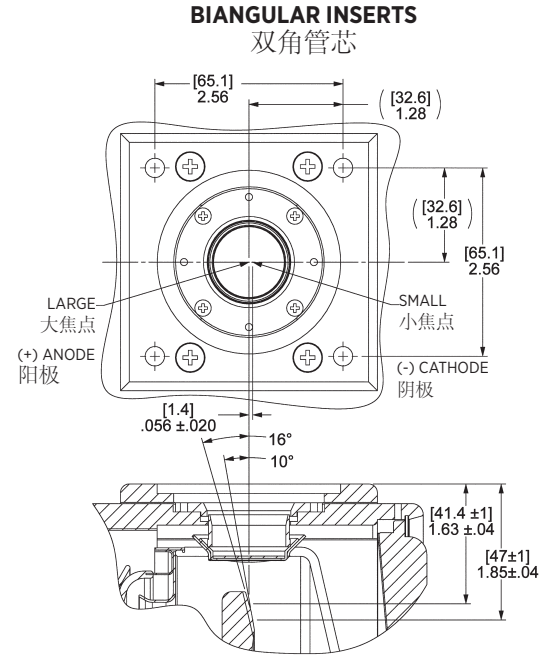Central Ray and Reference Axis
中心射线及参考轴

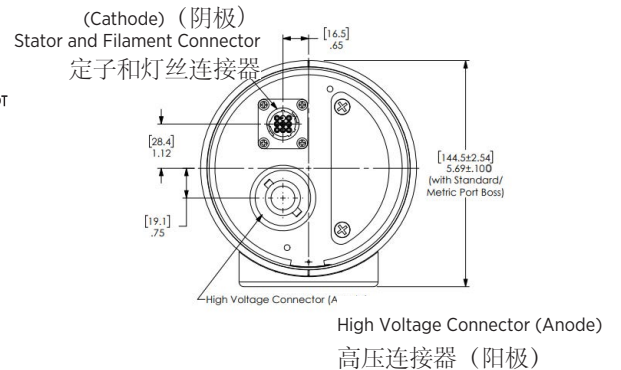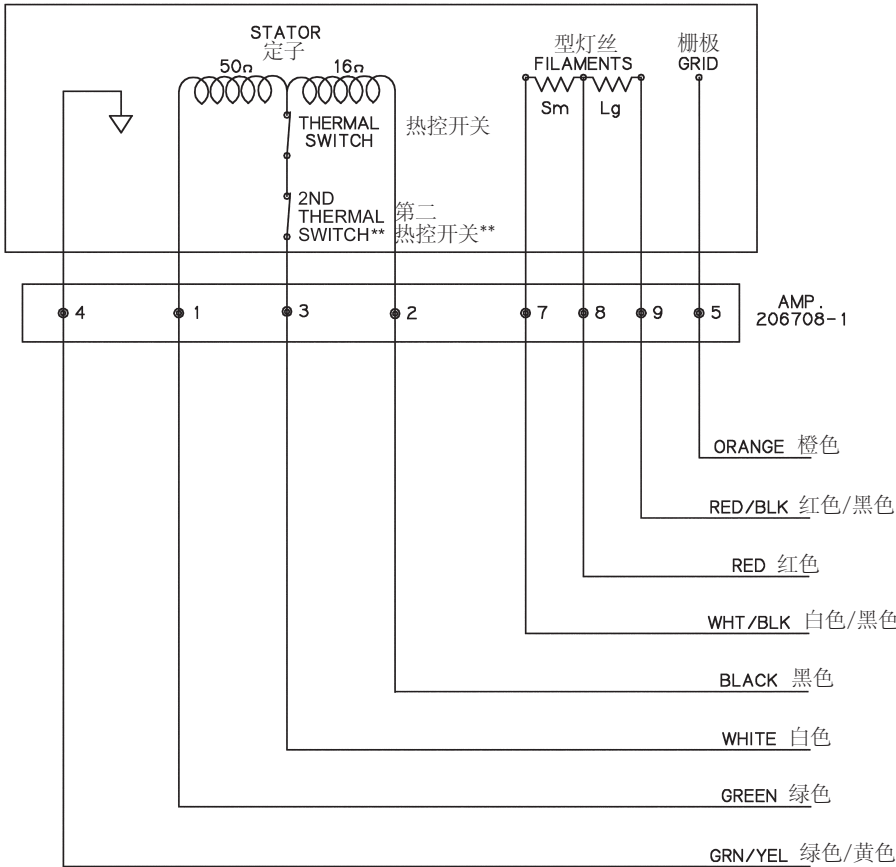Stator Ratings and Characteristics
定子额定值及特征

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Stator Power:
Time to full speed of the anode is a function of the power rating of the “starter” and the weight/diameter of the anode. All Varex Imaging stator types are rated for regular speed and high speed starters. Time to full speed of 71mm and 77mm anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible.

Starter:	Start	Run
50/60 Hz	115 VAC	45 VAC
150/180 Hz	230 VAC	90 VAC

NOTE:
For non gridded operation connect Grid Pin (Pin 5) to Filament Common (Pin 8).

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
Temperature Limits - Horizontal Applications
温度限制-水平应用

X-Ray Tube Assembly Heating and Cooling Curves

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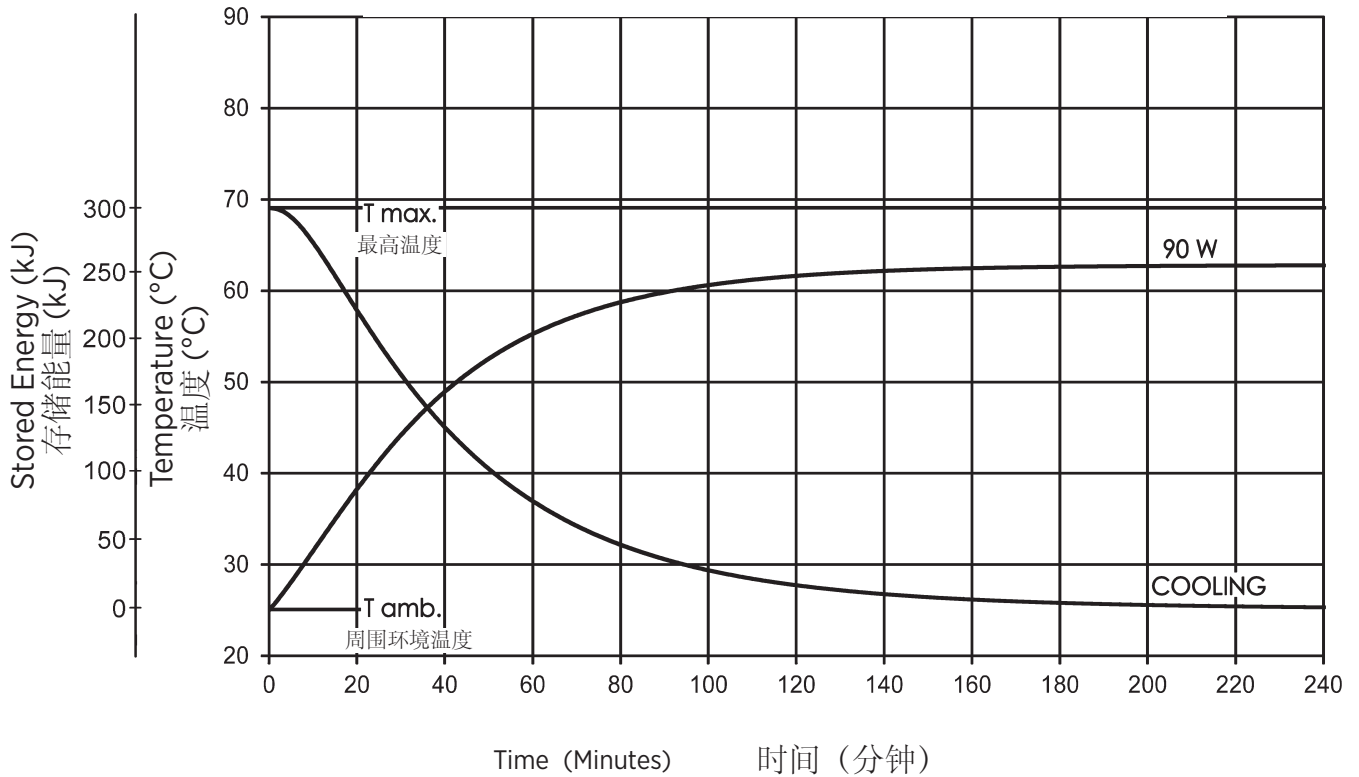
Temperature Limits - Vertical Applications
温度限制-垂直应用

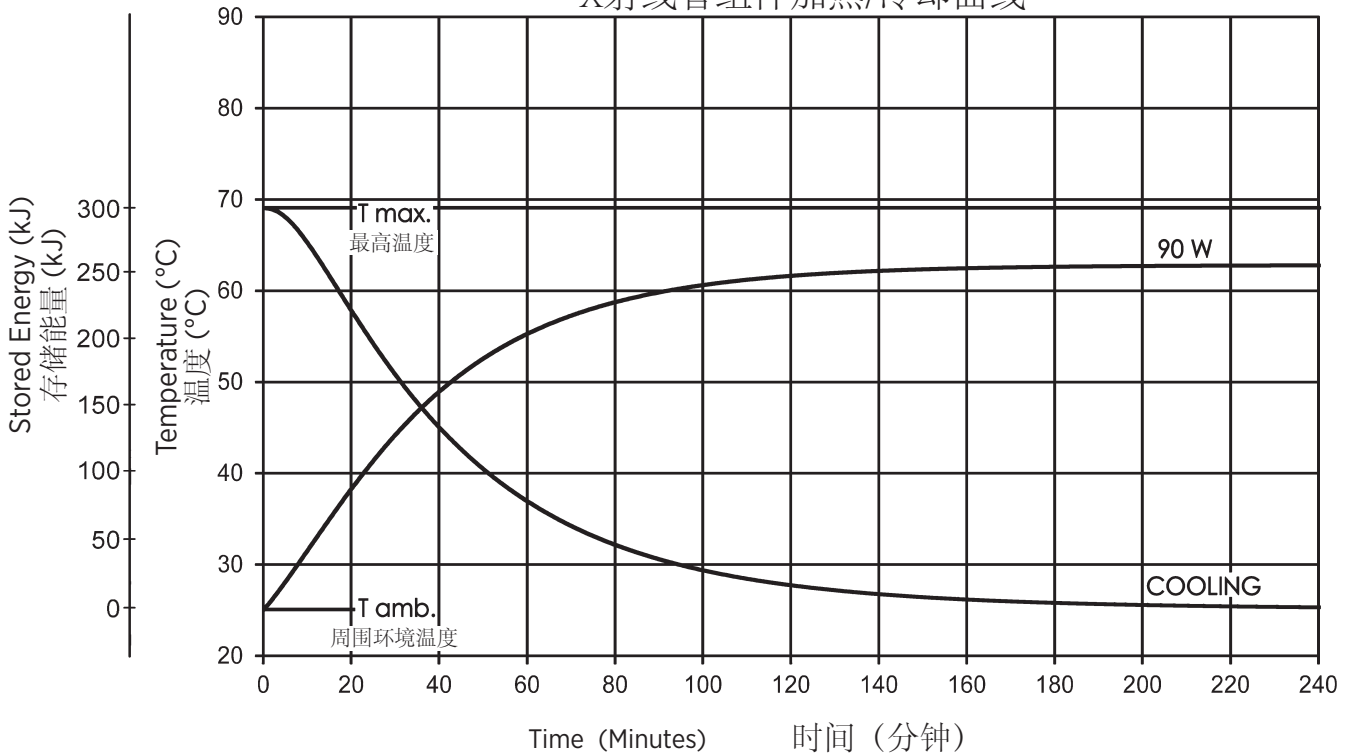X-Ray Tube Assembly Heating and Cooling Curves
X射线管组件加热/冷却曲线

Note:
Heat inputs into housing include tube power, filament power, and stator power. Heating curves based on no restrictions of natural convection around tube housing assembly.

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